


### Defibrillator Swing Arm

Find the section below which relates to your specific cart and follow the directions. Note the section on page 4 that relates to all carts.


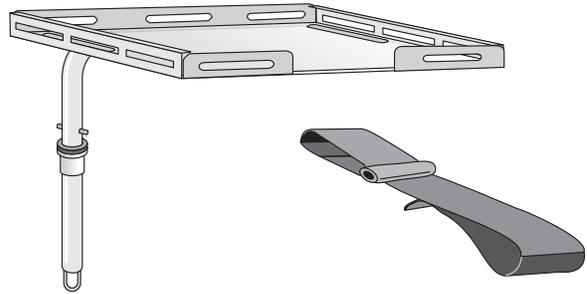


**CAUTION:**  
**WEIGHT CAPACITY**  
**40 lbs. (18.2kg)**


### MetroFlex™ Carts

Supplied Components include: (1) Defib Arm Assy, (1) mounting post, (1) 2 piece velcro strap, (4) barrel nuts, (1) security screw.

1. Attach post to the left rear corner of the cart with 4 supplied barrel nuts as shown below.
2. Pull back spring button and insert platform arm down into the mounting post until the nylon bushing is seated on top of the post. Rotate the platform until the spring button seats properly.
3. Insert security screw into the mounting post.

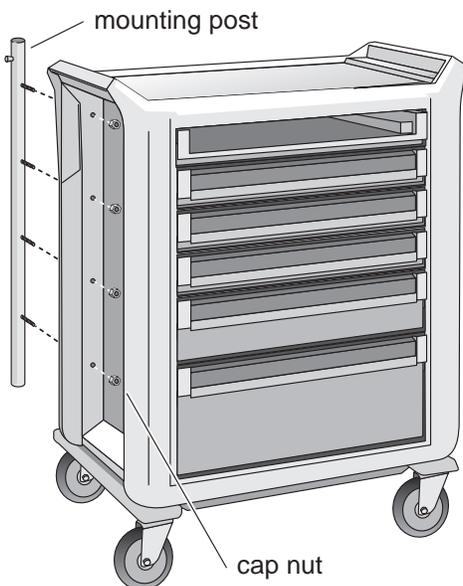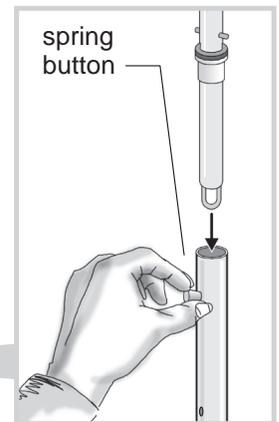

security screw

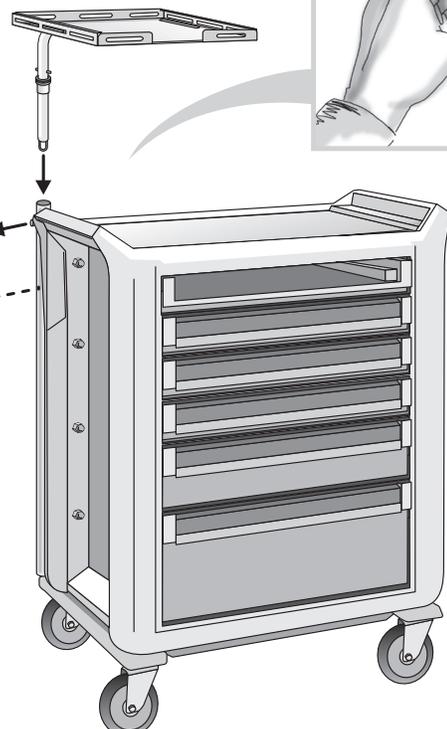


# Lifeline Carts

## If mounting post is already attached to cart...

If mounting post attached to cart contains Spring Button as shown, Supplied Components include:(1) Defib Arm assy, (1) 2 piece velcro strap, (1) security screw.


1. Remove post cap on mounting post on the rear left corner of the cart.
2. Pull back spring button and insert platform arm down into the mounting post until the nylon bushing is seated on top of the post. Rotate the platform until the spring button seats properly.
2. Insert security screw into the mounting post.

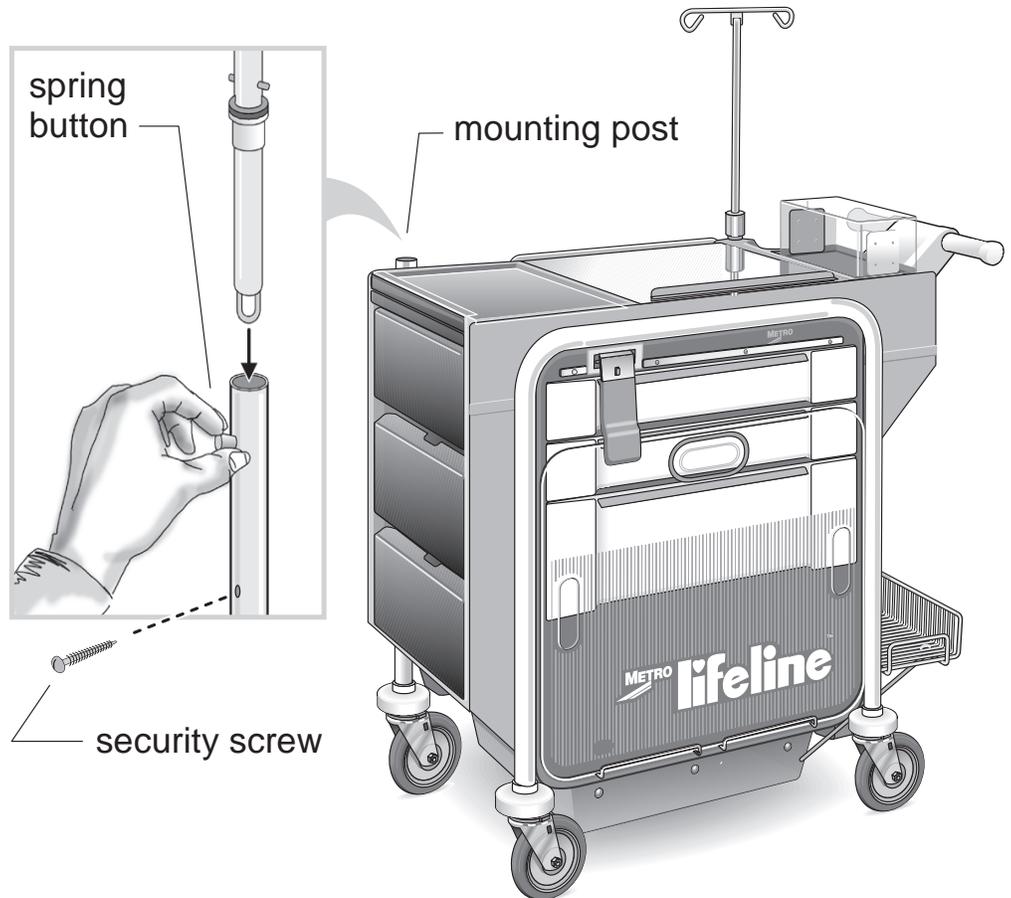

## If mounting post is NOT already attached to cart...

If mounting post attached to cart does not contain spring button as shown above, new post containing spring button must be used. Supplied Components include: (1) Defib Arm assy, (1) mounting post, (1) 2 piece velcro strap, (1) security screw.

1. Remove side bins and all three front drawers from the cart.
2. Remove the existing left rear post and caster. Detach the caster and bumper from this post and reinstall them in the new post.
3. Re-attach new post to cart using the original hardware.
4. Replace bins and drawers and attach the swing platform as described above.

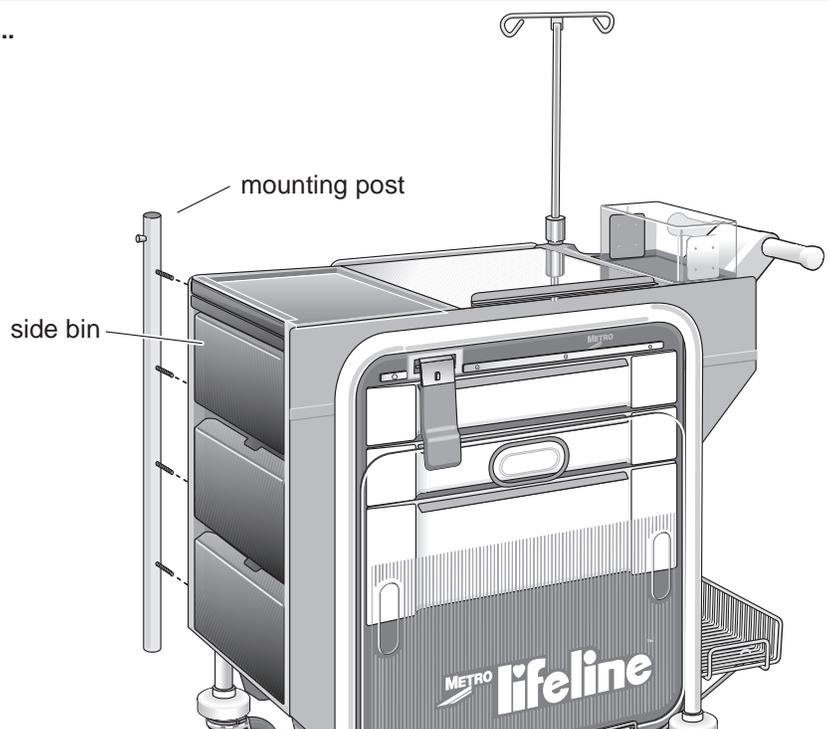


# Starsys™ Carts

## If mounting post is already attached to cart...

If mounting post is already attached to cart, Supplied Components include: (1) Defib Arm assy, (1) 2 piece velcro strap, (1) security screw, (1) security nut.




1. Pull back spring button and insert platform arm down into the mounting post on the rear left corner of the cart until the nylon bushing is seated on top of the post. Rotate the platform until the spring button seats properly.
2. Insert security screw into the mounting post and secure with nylok nut.

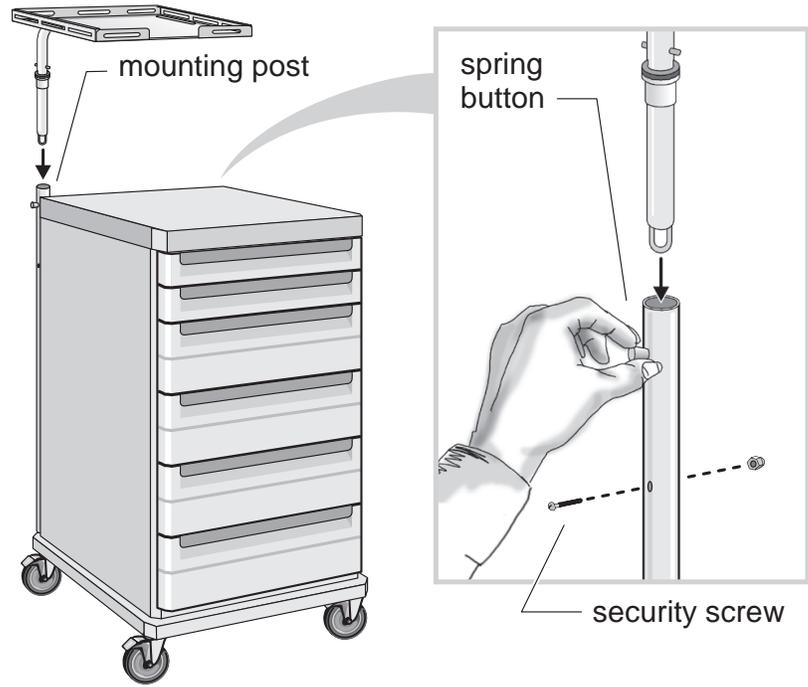

## If mounting post is NOT already attached to cart...


If mounting post is not mounted to cart, Supplied Components include: (1) Defib Arm assy, (1) mounting post, (1) 2 piece velcro strap, (4) screws, (1) security screw, (1) security nut.

1. Carefully cut two windows in the filler strip on the left rear post using the measurements shown at right. First score across horizontally as shown and then cut vertically between the scores. Break out and remove each section of filler strip.
2. Hold mounting post with brackets up to the corner post with the brackets aligned with the windows in the filler strip and mark the four hole positions inside the windows on the corner post.
3. Drill four holes through the corner post with a 9/64" (3.6mm) drill bit.
4. Use the supplied screws to attach the mounting post to the cart.
5. Attach the swing platform as described above.

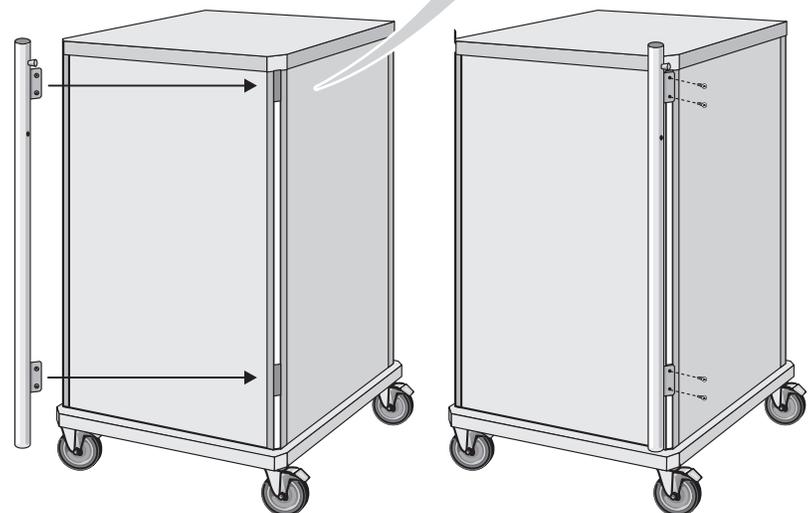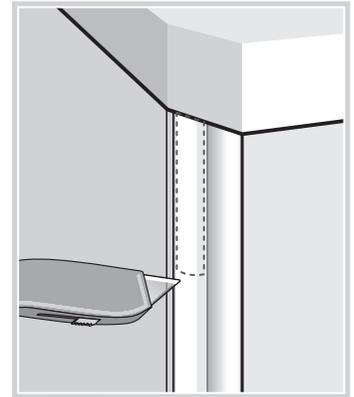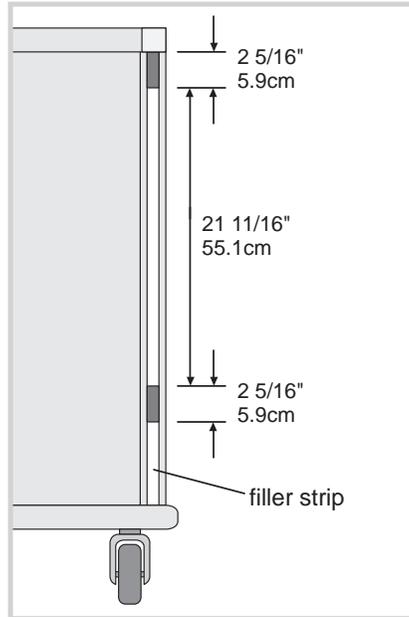


## All Units


Apply straps as required to secure defibrillator to platform.

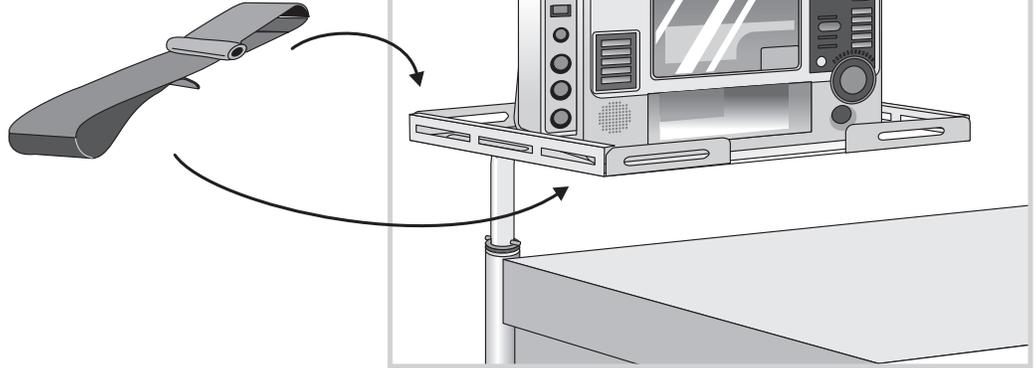


Swing platform into the positions shown at right by pulling and releasing spring button.

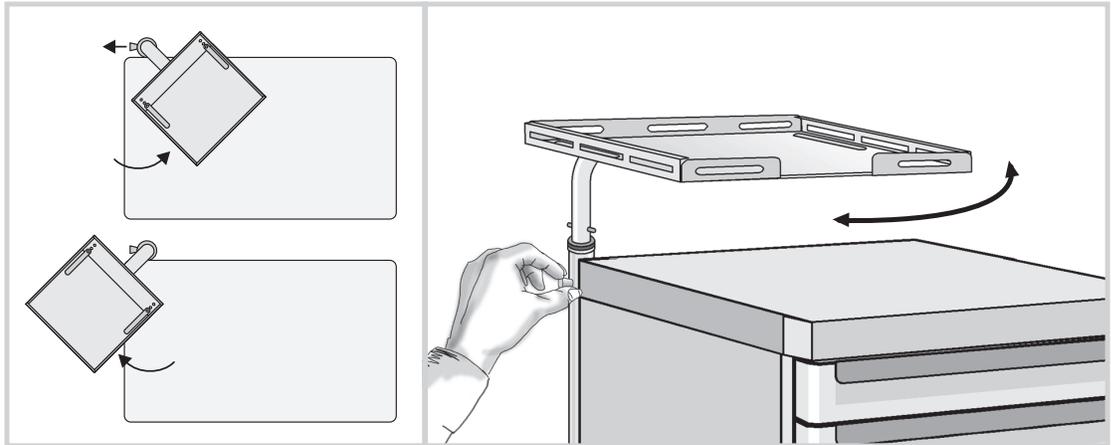

Adjust size of platform base by releasing slide buttons.

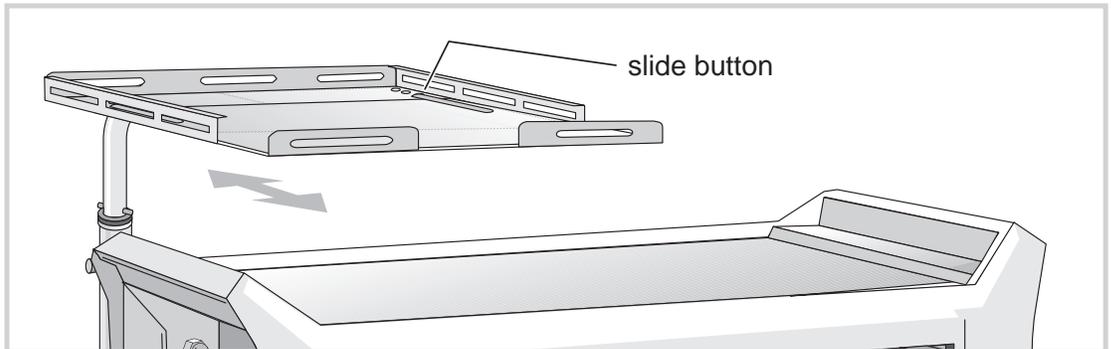

Platform rotates freely.

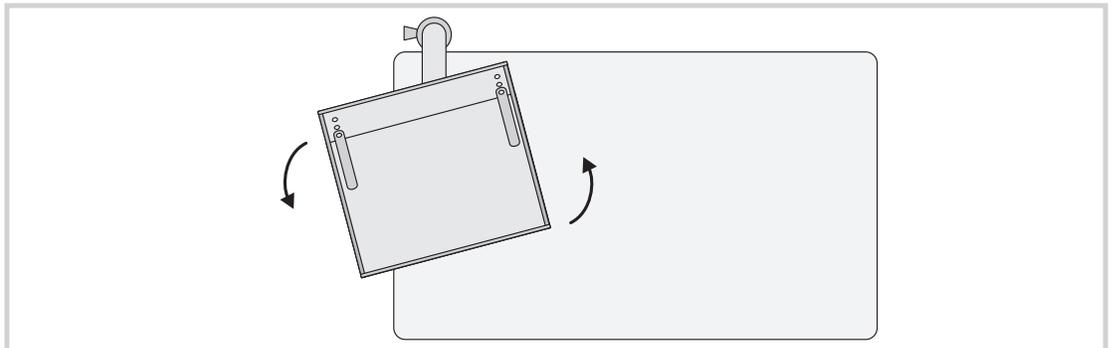

### InterMetro Industries Corporation

North Washington Street, Wilkes-Barre, PA USA 18705

For Product Information, Call 1-800-433-2232

Pour de plus amples renseignements sur les produits, appeler au 570-825-2741

Para obtener información sobre el producto llame al: 570-825-2741

L01-404 4/04

Information and specifications are subject to change without notice.  
Please confirm at time of order

Les données et caractéristiques techniques sont susceptibles d'être  
modifiées sans préavis. Veuillez confirmer au moment de la commande.  
Las informaciones y especificaciones están sujetas a cambios sin  
previo aviso. Por favor, confirme a la vez que realiza el pedido.