METRO STARSYS Overbridge

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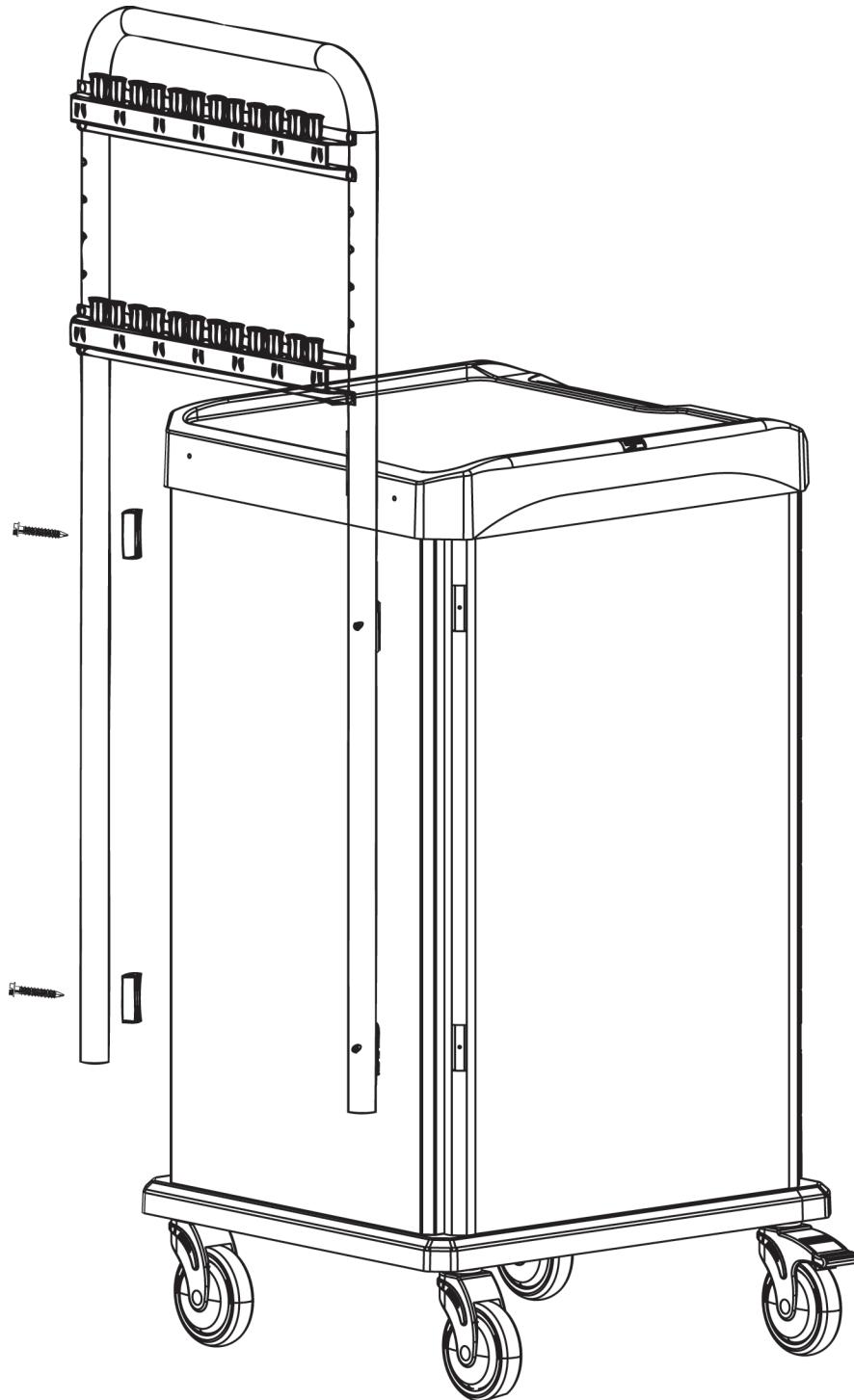
METRO STARSYS Puente

ASSEMBLY INSTRUCTIONS

INSTRUCTIONS D'ASSEMBLAGE

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METRO

InterMetro Industries Corporation

North Washington Street, Wilkes-Barre, PA 18705

For Product Information Call: 1-800-992-1776

Visit Our Web Site: www.metro.com

METRO STARSYS Overbridge

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METRO STARSYS Puente

LIST OF COMPONENTS

For the Assembly of the
Starsys Overbridge

PART	QTY.
Overbridge Support	1
Hanger Rail	2
Spacer	4
Overbridge Mounting Tapcon Screw #3/16 x 2 ³ / ₄ Lg.	4
Hanger Rail Screw #10/24 x 1 ¹ / ₂ Lg.	8
Hanger Rail Bushing Single Wide Unit	12

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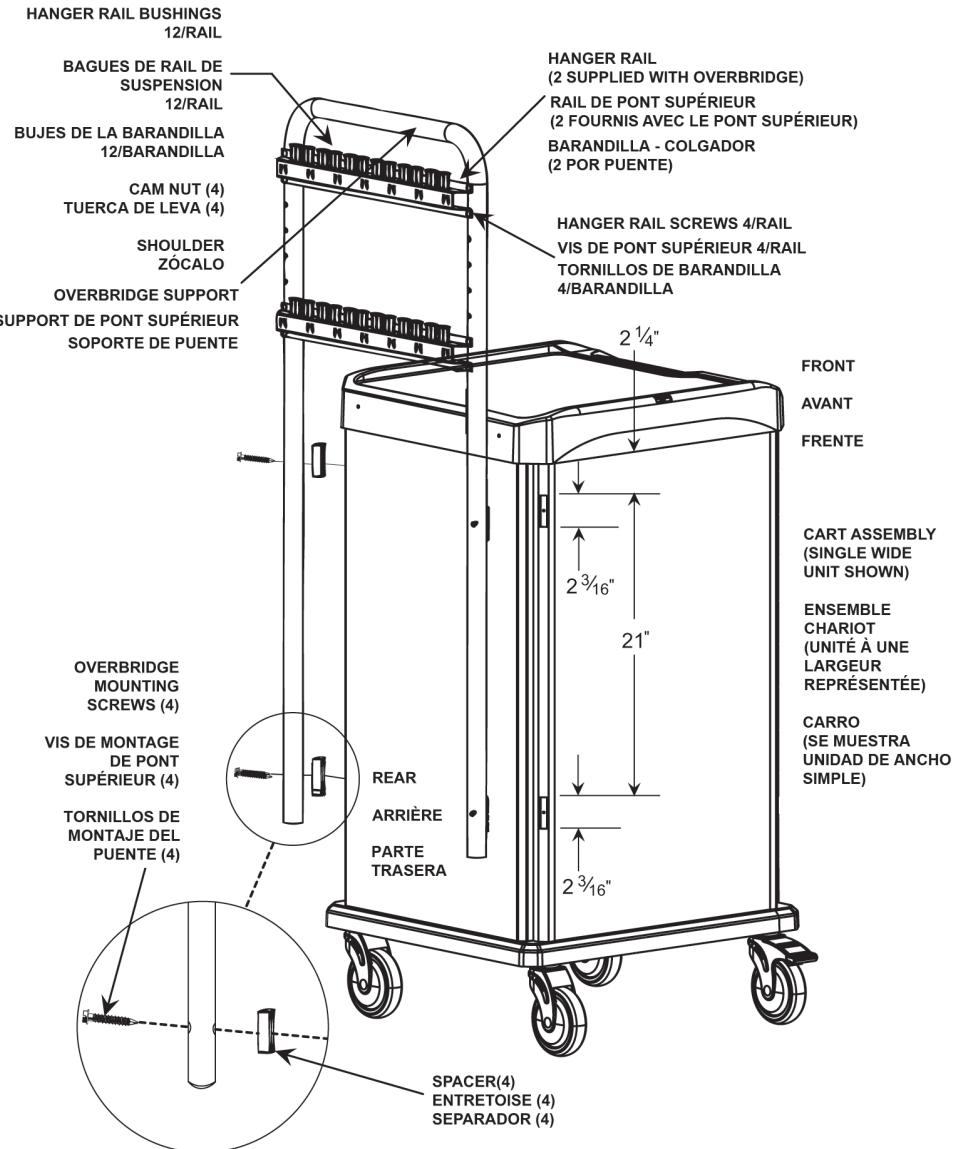
OVERBRIDGE ASSEMBLY INSTRUCTIONS

1. Unpack carton to confirm parts and quantities.
 2. If retro-fitting an existing cart with an Overbridge kit, the rear filler strips must have windows cut in them. Consult Drawing for measurements. It is recommended to score the strips horizontally then cut vertically between scores for all 4 cut outs. Break out each section. If cart filler strips are already cut, proceed to Step No. 3.
 3. Position the overbridge at desired location and mark the Starsys posts through the spacers to locate the 4 holes that must be drilled in to the Starsys posts.
 4. Drill a $\frac{1}{8}$ " diameter hole through the outer wall of the Starsys post and through both walls of the inner tube (do not drill through the opposite outer wall of the Starsys post).
 5. Repeat Step 4 for the other 3 mounting screw locations.
 6. Install a $2\frac{3}{4}$ " long tapcon screw through each hole in the over bridge/spacer and then into the drilled holes in the Starsys posts and tighten securely.
 7. Mount the hanger rail(s) to the back of the overbridge as shown. The flanges of the rail front must fit flat against the overbridge support. The L-shaped slots spaced along the length of the rail must be with wide end up as shown. The hanger rail(s) may be mounted at various positions on the front or back of the support. Secure each rail with the supplied #10- $\frac{1}{2}$ screws.
 8. Install the bushings by pushing each one down into the rail opening until they snap in place. The bushing tab must face the open side of the rail.
- NOTE:** Hanger rail bushings must be used to obtain proper fit and to protect the finish on the rail and accessories.

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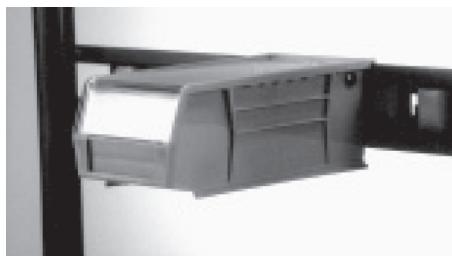
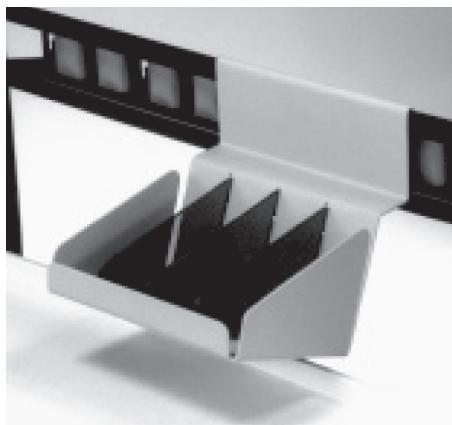
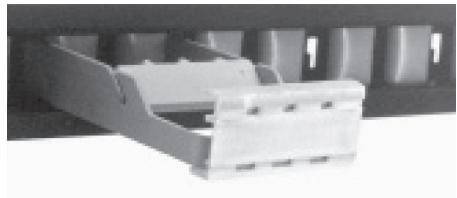
The Label/Tape Dispenser provides easy one-handed access to rolls of identification labels and tape. It **MUST** be mounted into the recessed front of the hanger rail as shown.

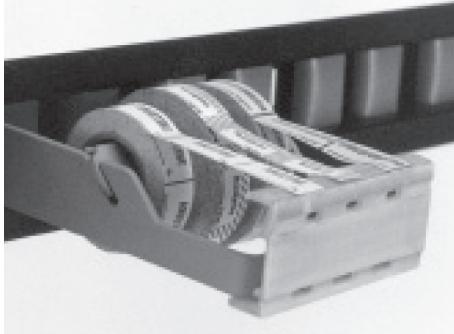
To install the Dispenser, remove the core and gently compress the Dispenser side arms and insert the tabs through two L-shaped openings. Release the pressure and the arms will spring apart into the wider portion of the opening. When properly installed, the slotted-tabs will rest on the upper ledge of the L-shaped opening, hooking behind the Rail.

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Remove and place spools of tape or labels onto the core. Orient printed matter so that it will be readable, and so that the material feeds forward to the cutting bar from the top of the roll. Drop the core and spools into the slots so that the core is in the furthest back position. Test the Dispenser to insure a solid installation and proper performance.

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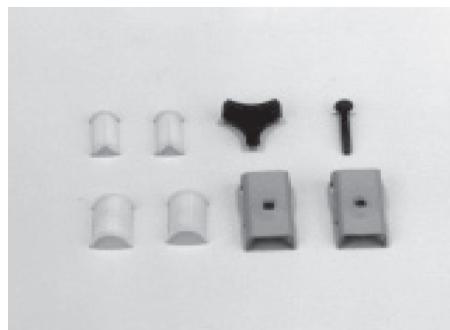
No. de catalogue SX570

Pinza universal:

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One or more Universal Clamp may be positioned wherever needed on the Overbridge. The Universal Clamp can support the Utility Pole, Laryngoscope Blade/Specimen Bag Holder, the Sharps Container Bracket, Short and Long Hooks.

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The plastic sleeves clip into the metal clamp halves. Snap the larger sleeve halves into the large recesses of the metal clamp. Squeeze the small sleeve halves together and install them into the front end of the clamp. Position the larger openings in the metal clamp around the Overbridge as shown. Insert the Carriage Bolt through the Clamp making sure the flats of the bolt head fit solidly into the square hole. Thread the knob onto the exposed bolt shaft and tighten to secure.

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METRO STARSYS Overbridge

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METRO STARSYS Puente

To mount an accessory, loosen the knob, insert the accessory into the Clamp's small hole, position the Clamp and tighten the knob to hold in place.

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Wire Supply Baskets can be hung on the back of the Rail hanging in the bushings or off the front using the holes provided. **NOTE:** Three positions are possible on the front.

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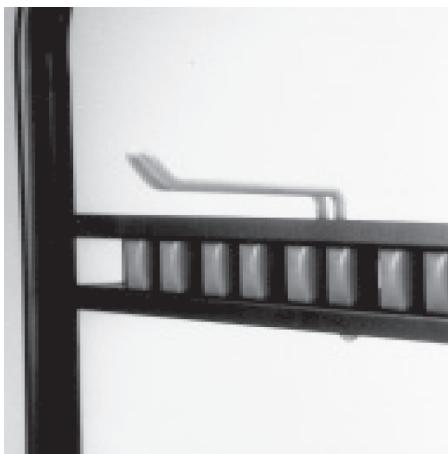
Compress the hook base to insert it into the rail bushing of your choice.

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Laryngoscope Blade/Specimen Bag Holder:
Catalog No. SX575

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Compress the base to insert it into the rail or bushing of your choice. The Laryngoscope Blade/Specimen Bag Holder can also be used with the Universal Clamp accessory. **NOTE:** Standard gallon size zip lock poly bags can be used to transport scope/specimen to other locations.

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InterMetro Industries Corporation

North Washington Street, Wilkes-Barre, PA 18705

For Product Information Call: 1-800-992-1776

Visit Our Web Site: www.metro.com

L01-244
REV E
03/11

Information and specifications are subject to change without notice. Please confirm at time of order.