
M400LS DRAWER CHANGE

1. Remove storage drawers and patient drawers from left side of cart.
 - a. Remove storage drawer by pulling the drawer out then locate the release lever on each side of the drawer slide. Push down on one side and pull up on the opposite side of the release lever. Pull with a slight tug and remove the drawer from the slide.


- b. Remove shelf if there is one.
2. Install slide pairs onto side of cart body as shown in photo. Fasten with 8-32 x 1/4 round head screw.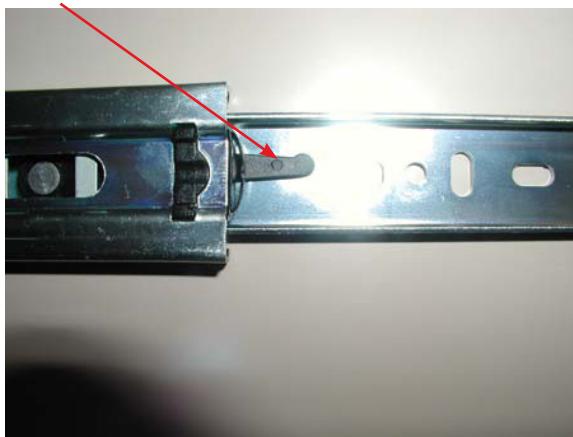


M400LS DRAWER CHANGE— LIONVILLE SYSTEMS MEDICATION CART

- a. Fasten top slide pair in the 1st slide position.
- b. Fasten 2nd drawer slide in the 3rd slide position. Use upper set of holes.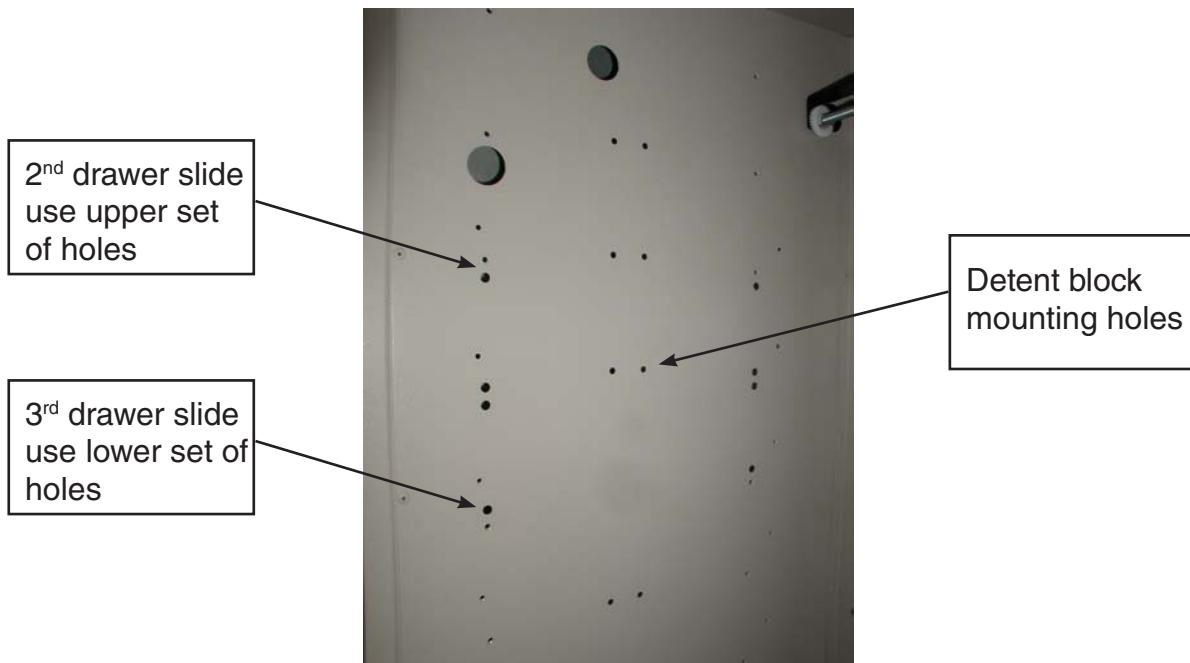

- c. Fasten 3rd drawer slide pair in the 6th slide position. Use lower set of holes.
 - d. Fasten 4th drawer slide pair in the 9th slide position.
3. Install new thick spacer plate above third drawer and secure with 6-32 x 1/4 round head screws.

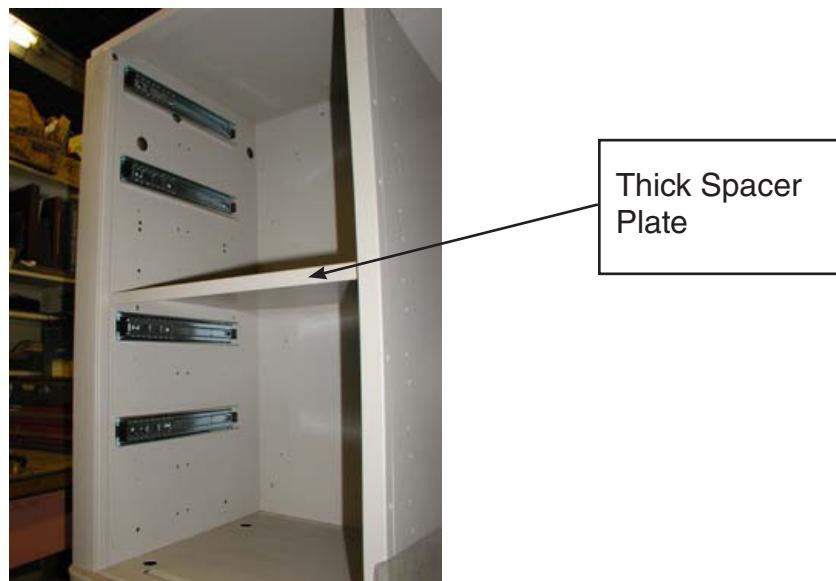M400LS DRAWER CHANGE — LIONVILLE SYSTEMS MEDICATION CART

4. Install detent block left side wall for all storage drawers.

a. If there are no preexisting holes.


- I. Using template provided. Mark and drill two holes for the detent block with a #29 drill bit.
- II. Fasten detent block with #8 sheet screws.

b. If there are preexisting holes.

- I. Install #6 threaded inserts below each slide pair with appropriate installation tool.
- II. Fasten detent block to wall with 6-32 x 5/8 " pan head screw.

5. Install striker block onto right side wall below slide for each CS drawer installed.

a. Fasten with 6-32 x 3/8 round head screw.

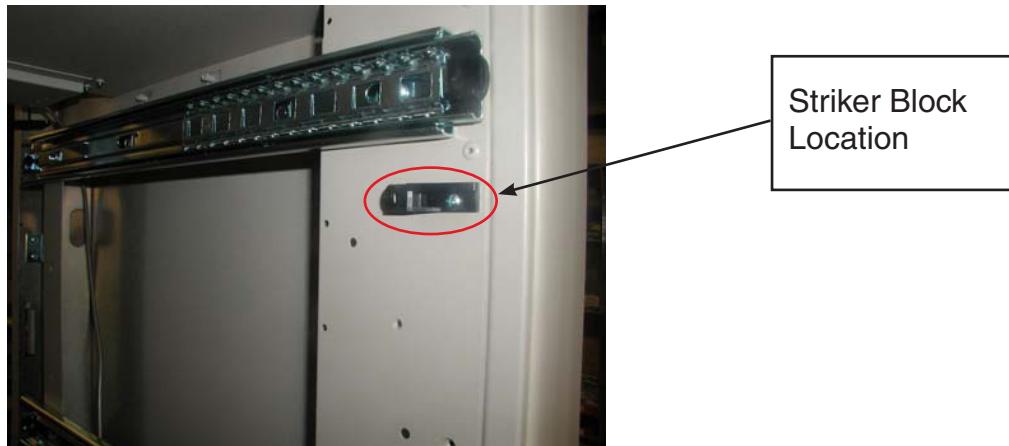6. Place storage drawers back into cart.

- a. Install 2 tier CS storage drawer in the top slide position. The catch on the back will need to be in the lower set of mounting holes.
- b. Install 3 tier CS storage drawer in the 3rd slide position. The catch on the back for this drawer needs to be in the lower set of mounting holes.
- c. Install 2 tier storage drawer in the 6th slide position. The catch will need to be in the upper set of holes.

- d. Install 3 tier storage drawer in the 9th slide position . The catch for this drawer will need to be in the upper set of mounting holes.
- 7. Check manual lock operation of cart. CS drawers will unlock after the cart is unlocked.
- 8. The configuration should look like the photo below.

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© 2023 InterMetro Industries Corporation, Wilkes-Barre, PA 18705