



LIONVILLE® CARTS-ADJUSTABLE CUP DISPENSER

About Dispensers and Cups:

1. Built-in cup dispensers are available in 2 sizes:
 - (1) a 2-1/2" wide med and
 - (2) a 3-1/2" wide water cup.
2. The Medication dispenser holds 3/4 to 1 oz. soufflé or 1 oz. plastic liquid med cups. The water dispenser holds 3 oz. to 8 oz. cold liquid cups. Foam cups won't fit into the water dispenser.
3. Place cups in dispensers with the open end down. Follow the instructions below to adjust the dispenser to your cups.

To Adjust Dispensers:

1. Pull the dispenser up and out of the cart - it will be snug but can be removed. If the dispenser comes apart see directions below for reassembling it.
2. Place a cup in the dispenser tube (open end down) then twist the top collar of the dispenser until the plastic band ("O" ring) catches the edge of the cup (don't make it too tight). Pull the cup out to be certain that the "O" ring is properly adjusted. Put several cups in and pull them out to be certain that the grip isn't too tight.
3. After dispensers are adjusted, you won't have to readjust them unless you change to a different cup.

To Reassemble a Cup Dispenser:

1. Place the spring in the tube.
2. Put the spring cap over the spring (open side down). Push it down to hold the spring in the tube.
3. Locate the 3 tabs on the tube that are slightly rounded on the outside. Hold the cap down and place the "O" ring over these 3 tabs (it will form a triangle inside the tube).
4. Position the dispenser collar so that its 3 tabs are directly in front of the tabs holding the "O" rings.
5. Snap the collar in place.

* Save this document for future application, load rating and/or safety reference.

© 2019 InterMetro Industries Corporation, Wilkes-Barre, PA 18705
L07-094 REV C 10/19



We put space to work.®

an Ali Group Company



The Spirit of Excellence