


Case Cart Static Dissipative Cable

- 1) The supplied drag cable kit can be installed in various locations on the underside of the case carts:
 - a. The cart has several 0.25"(6mm) diameter mounting holes on the underside support channel to mount the cable, or
 - b. If the cart underside does not have 0.25" (6mm) diameter mounting holes, holes must be drilled in the underside support channel. Refer to procedure 3-B, take care to choose a location that will not interfere with the wheels.
- 2) Take care to place the cable under the cart where it won't interfere with your washing equipment or other items in your facility.
- 3) It is recommended that after the cable is mounted, the case cart be operated through the entire hospital operation (including the cart wash) before first use.

Kit Contains:




Drag Cable 14" (356mm) Long (**Qty 1**)
 #10-24 x ½ Stainless Steel Phillips Pan Head Screw (**Qty 1**)
 #10-24 Stainless Steel Lock Nut (**Qty 1**)
 #10 Stainless Steel Flat Washer (**Qty 1**)

Assembly Procedure:

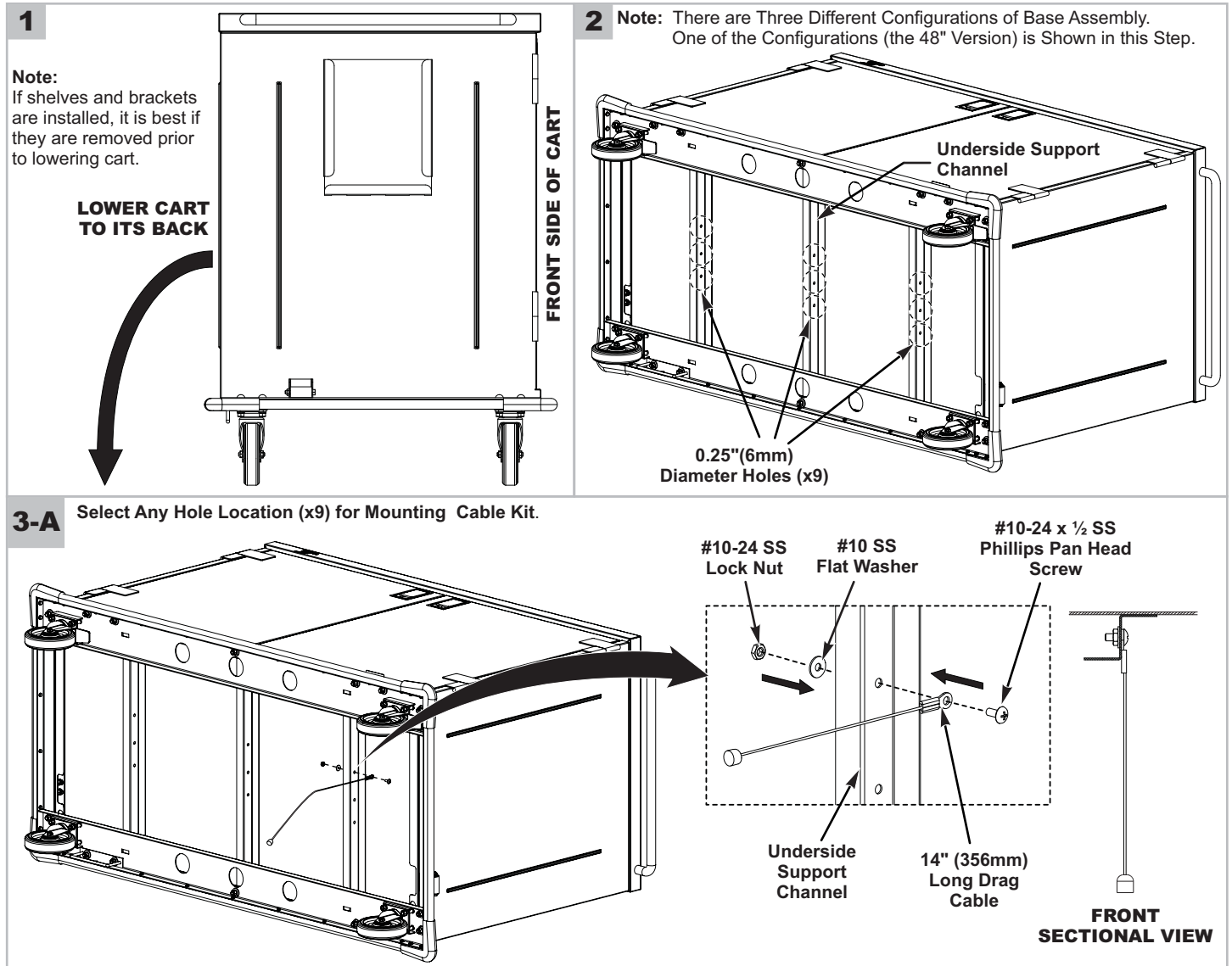

Case Cart Static Dissipative Cable

3-B FOR CARTS WITHOUT PRE-PUNCHED HOLES

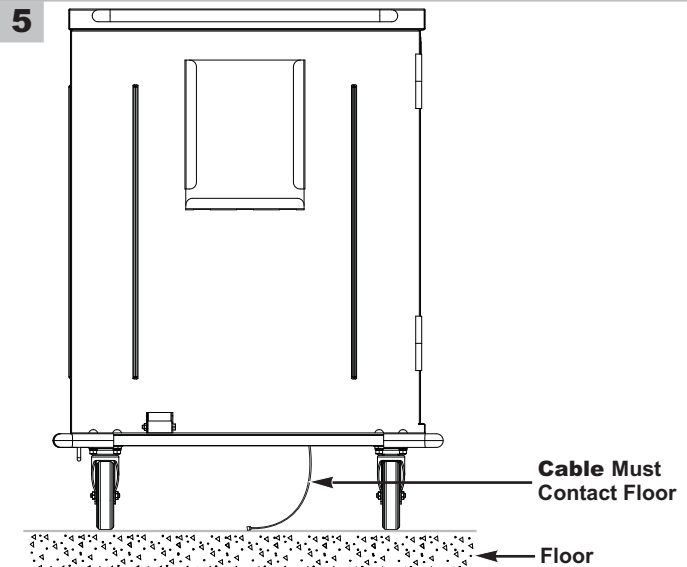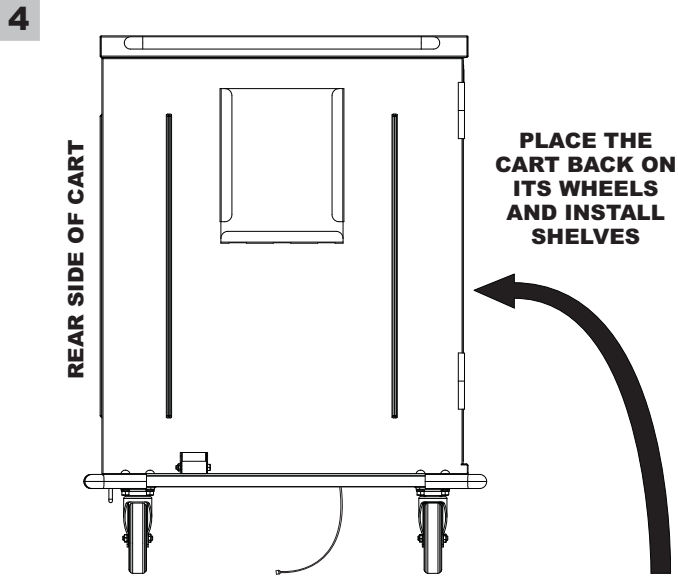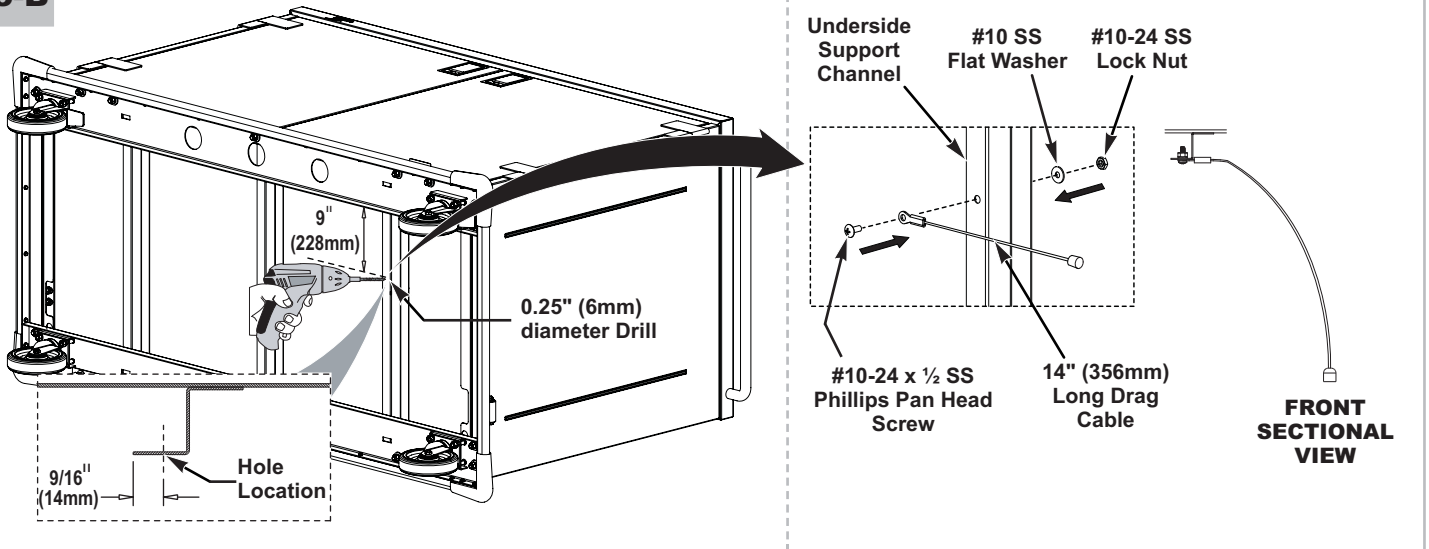

* Save this document for future application, load rating and/or safety reference.



InterMetro Industries Corporation
North Washington Street, Wilkes-Barre, PA USA 18705
For Product Information Call: **1-800-992-1776**
Visit Our Web Site: **www.metro.com**

INST-CASECBL
05/2014

Information and specifications are subject to change without notice. Please confirm at time of order.